

# LINELASE™

## (STRIAE)

### DYNAMIS

© 2023 Fotona, LLC. All rights reserved.

---

#### PRE-TREATMENT INSTRUCTIONS

##### 2 Weeks Prior

- Avoid sun-exposure, tanning beds, or sunlamps. Patients are unable to be treated with a tan or a sunburn in the face.
- Avoid perfumes, deodorants, waxing, chemical peels or collagen injections in the treatment area.

##### 1 Week Prior

- Discontinue taking light sensitive medications.

##### 3 Days Prior

- Discontinue the use of Aspirin. This can increase the risk of bleeding.
- 

#### POST-OPERATIVE INSTRUCTIONS

##### General

- Apply a re-epithelizing cream as often as needed to improve the healing process. Cream recommendations to select from are:
  - Bariederm Cicacream (Uriage)
  - Cicaplast (La Roche Posay)
  - Cicalfate (Avenne)
  - Aquaphor (Eucerin)
- Apply broad spectrum sunscreen, 3 times a day (7 am, 12 pm and 2 pm).

##### For 3 Days

- Wear loose clothing for a few days to avoid friction and pain.

##### For 7-15 Days

- Strict avoidance of sunlight.
- 

#### HEALING AND RESULTS

- Edema, erythema, purpura, bullae or crusts can be expected to last 7 to 15 days, depending on aggressiveness of the treatment.