

SEBACEOUS HYPERPLASIA

DYNAMIS

© 2023 Fotona, LLC. All rights reserved.

POST-TREATMENT INSTRUCTIONS

General

- Apply a re-epithelizing cream as often as needed to improve the healing process. Cream recommendations to select from are:
 - Bariederm Cicacream (Uriage)
 - Cicaplast (La Roche Posay)
 - Cicalfate (Avenne)
 - Aquaphor (Eucerin)
- Apply broad spectrum sunscreen, 3 times a day (7 am, 12 pm and 2 pm).

Immediately After

- Can apply make-up (but best to wait 24 hours).

After 2-3 Days

- Topical treatments and skin care dermo-cosmetic products, such as retinol and acids can be resumed.

For 7 Days

- Strict avoidance of sunlight.
-

HEALING AND RESULTS

- After the procedure, a small scab is expected for at least 5-7 days.