

SKIN TAGS

DYNAMIS

© 2023 Fotona, LLC. All rights reserved.

PRE-TREATMENT INSTRUCTIONS

2 Weeks Prior

- Avoid sun-exposure, tanning beds, or sunlamps. Patients are unable to be treated with a tan or a sunburn in the face.
- Avoid perfumes, deodorants, waxing, chemical peels or collagen injections in the treatment area.

3 Days Prior

- Discontinue the use of Aspirin. This can increase the risk of bleeding.
-

POST-TREATMENT INSTRUCTIONS

General

- Apply a re-epithelizing cream as often as needed to improve the healing process. Cream recommendations to select from are:
 - Bariederm Cicacream (Uriage)
 - Cicaplast (La Roche Posay)
 - Cicalfate (Avenne)
 - Aquaphor (Eucerin)
- Apply broad spectrum sunscreen, 3 times a day (7 am, 12 pm and 2 pm).

For 5-7 Days

- In patients with darker skin, use a topical soft steroid to improve the healing process and avoid PIH, twice a day.
 - Prednicarbate cream 0.25%
 - Methylprednisolone Aceponate cream 0.01%

For 7 Days

- Strict avoidance of sunlight.
-

HEALING AND RESULTS

- After the procedure, a small scab is expected for at least 7 days.